


ASSIST BAR INSTALLATION INSTRUCTION – ZENITH BED

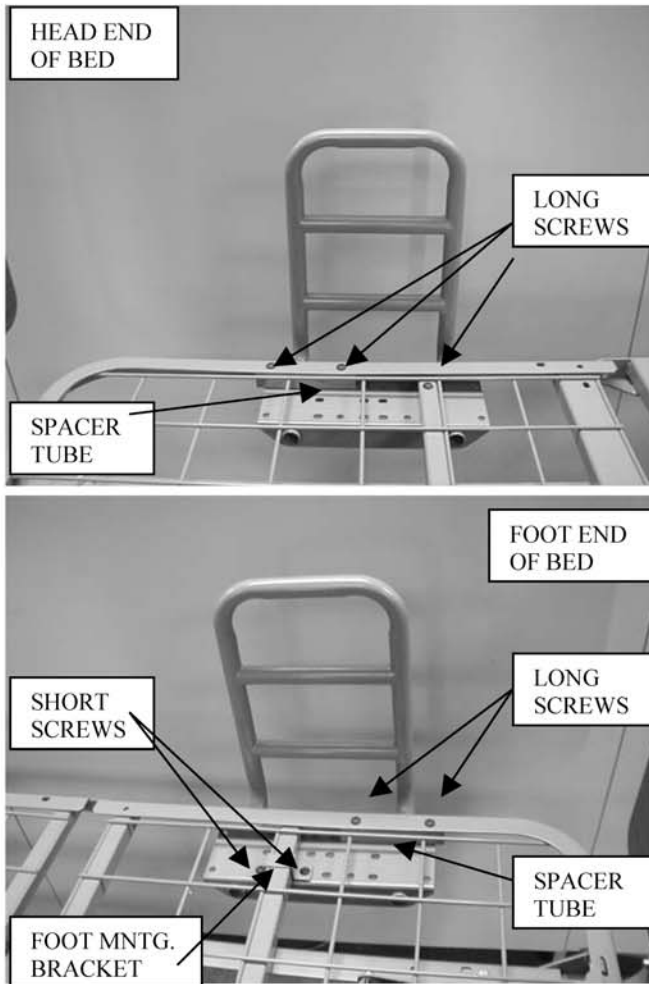

HEAD END OF BED:

1. Orient Assist Bar as shown.
2. Locate Spacer Tube under mattress deck and above Assist Bar Bracket, and fasten using two Long screws and lock nuts (Lock nuts should be hand-tight only).
3. Locate slot in cross tube and fasten with Long screw and nut.
4. Tighten all fasteners securely.

FOOT END OF BED:

1. Orient Assist Bar as shown.
2. Locate Spacer Tube under mattress deck and above Assist Bar Bracket, and fasten using two Long screws through mattress deck angle, and hand-tighten lock nuts.
3. Orient Foot Mounting Bracket over cross tube and wire grid as shown. Secure Foot Mounting Bracket with two Short screws and lock nuts.
4. Tighten all fasteners securely.

Parts Included:

999-0735-901T Assist Bar	1
999-0735-004T Spacer Tube	1
999-0717-007T Foot Mntg. Brkt.	1
100-5431-036 5/16 x 1-3/4" Long Screws	3
100-5431-028 5/16 x 3/4" Short Screws	2
100-6731-006 5/16 Lock Nuts	4

#22-7350

