




## OPTIONAL PIVOT ASSIST BAR INSTALLATION—HEAD END ONLY (KIT 95-90350: INCLUDES ONE ASSIST BAR AND HARDWARE)


### Installation

1. On the head end, align the two holes in the Pivoting Assist Bar (#2) bracket with the two holes in the mounting bracket on the underside of the Head Deck (#1).
2. From the outside, insert the two 4.00" Clevis Pins (#3) through the holes in the Pivoting Assist Bar bracket and Head Deck mounting bracket.
3. From the inside, insert a 2.375" Hair Pin Clip through the small holes at the ends of each of the Clevis Pins to secure.

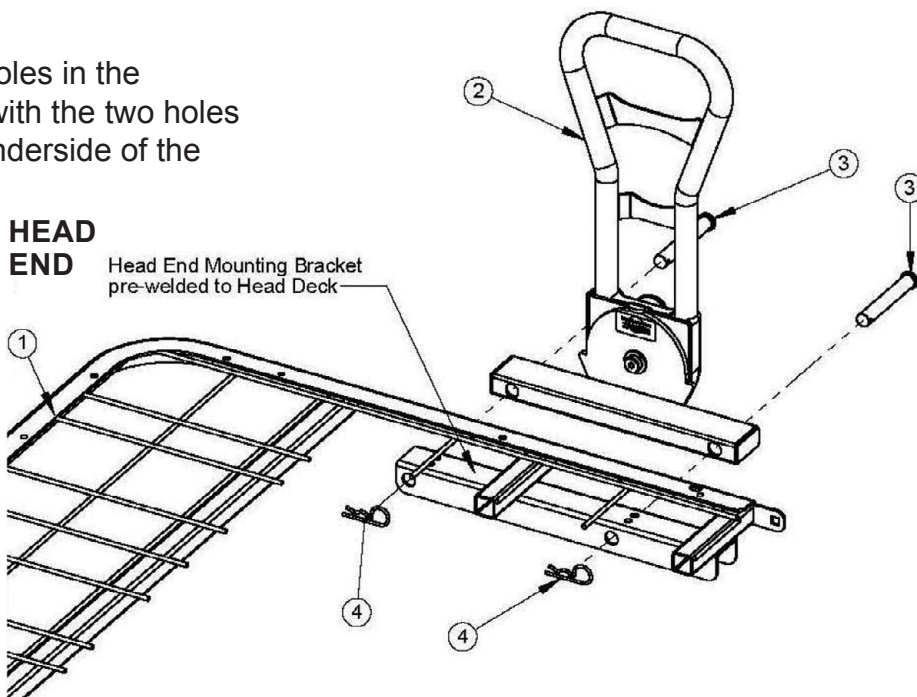

### Operation

1. To release the Pivoting Assist Bar from its vertical lock position, hold onto the top of the bar with one hand and slightly pull out the black knob on the outside of the assembly with the other hand to release the locking mechanism. For your safety, make sure your fingers are not near the pivot assembly when pivoting the rail. For ease of patient access, it is recommended that you pivot the rail toward the headboard until it stops and rests in place.
2. To set the assist bar back to its upright locked position, grab the top of the assist bar with one hand and pivot the assembly upward until the black knob mechanism snaps into place, locking the assist bar vertically.

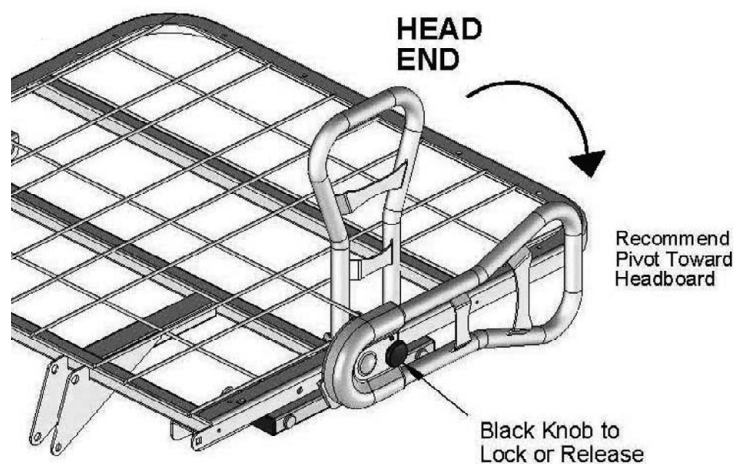

**!** TO AVOID PINCHING FINGERS, ENSURE FINGERS ARE CLEAR OF SIDES OR UNDERSIDES OF THE PIVOTING ASSEMBLY WHEN POSITIONING THE ASSIST BAR UP OR DOWN.

**!** TO AVOID PERSONAL INJURY, ENSURE THE PIVOTING ASSIST BAR IS ALWAYS IN THE FULL, VERTICAL, LOCKED POSITION WHENEVER RESIDENT/PATIENT IS LEFT UNATTENDED.

**!** POWERED AIR MATTRESS SURFACES MAY POSE A RISK OF ENTRAPMENT. PRIOR TO USE, ENSURE THE THERAPEUTIC BENEFITS OUTWEIGH THE RISK OF ENTRAPMENT.