




Parts List

Joerns® Bed Frames

Care 100

To avoid injury, read user's manual before using.

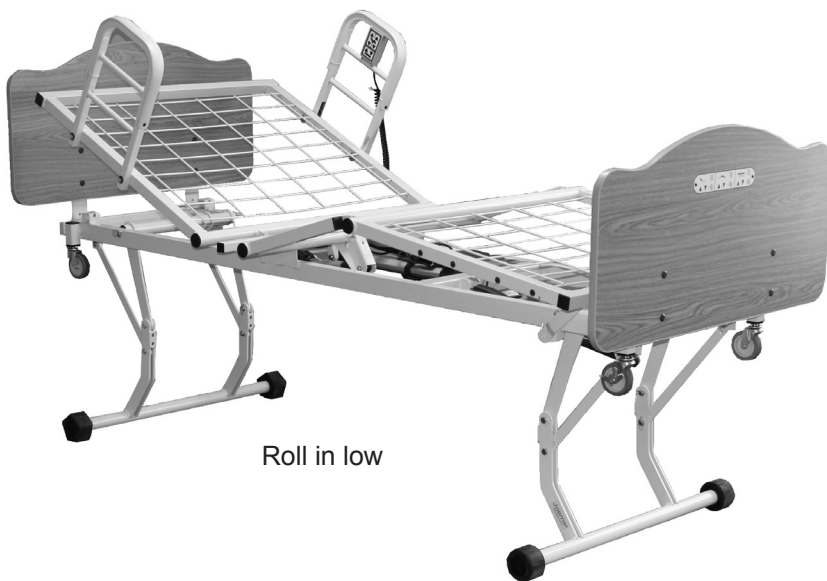


Roll in low



Roll at any height

FRAME & HIGH/LOW
MECHANISM COMPONENTS
80" FRAME
MODELS: CARE 100

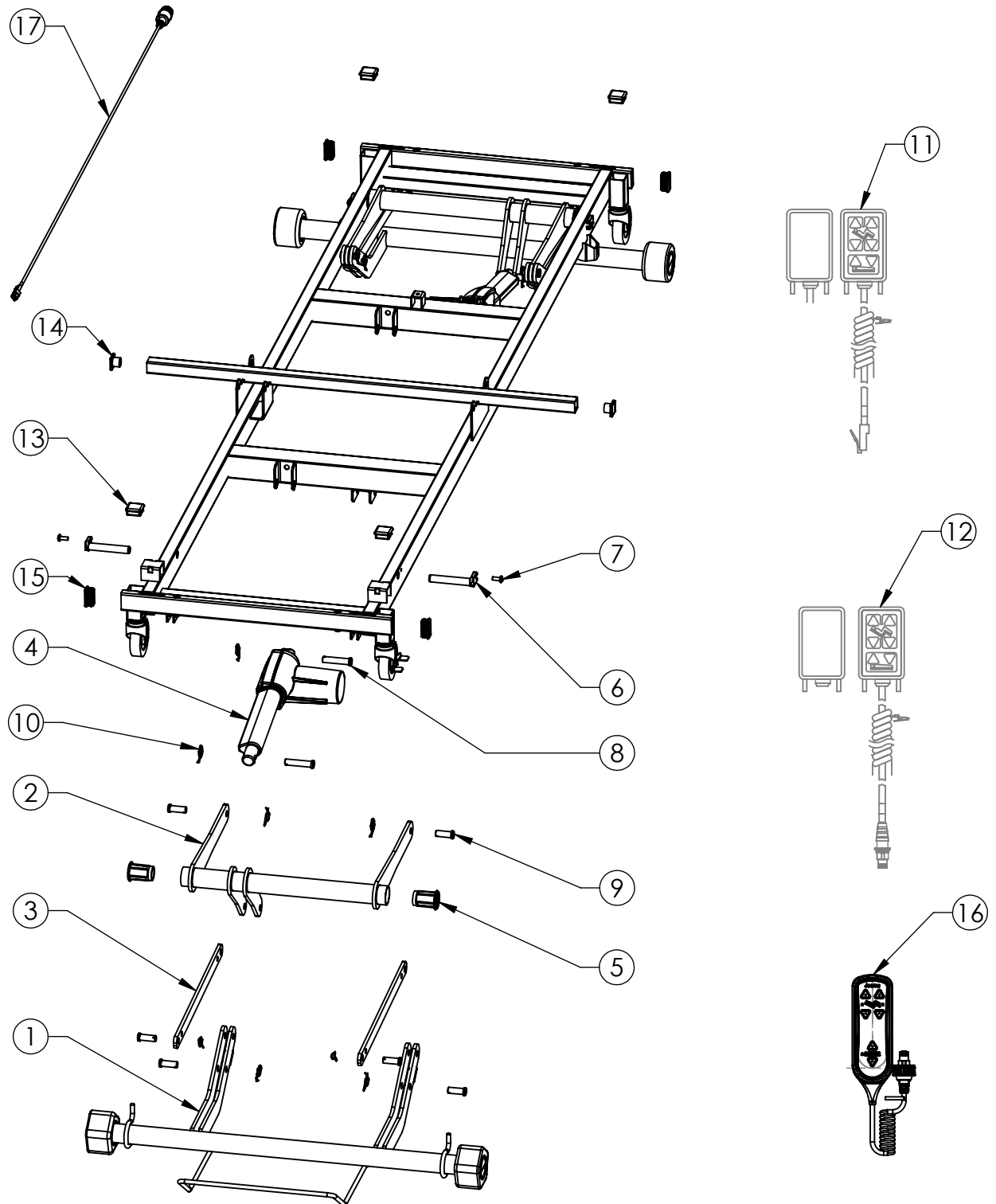


FRAME AND HIGH/LOW
80" FRAME
MECHANISM COMPONENTS
MODELS: CARE 100

ITEM NO.	PART NUMBER	DESCRIPTION
1	24006215	WELDMENT, LIFT, BOTTOM
2	24006390	WELDMENT, LINK, UPPER
3	11013678	LINK, CONNECTING
4	39001089	KIT, MATCHED SET ACTUATORS, ECC HI/LOW, LINAK
5	11011230	BUSHING, STABILIZER, PIVOT SHAFT
6	11012662	STABILIZER, PIVOT SHAFT
7	12552592	SCREW, 1/4-20 X 5/8", TX TRS HD
8	11012387	PIN, CLEVIS HEADED
9	11012388	PIN, CLEVIS HEADED
10	12800032	RING, RUE-22
11	11011474	PENDANT, 6-FUNCTION, PHONE CONN.(STAFF CONTROL)
12	11012262	PENDANT, 6-FUNCTION, CB6
NS	11013287	MOTOR CABLE, HI/LO ACTUATOR, RED O-RING
13	12470453	CAP, 1 1/4"X 1 1/4" SQUARE PLUG
14	12470467	CAP, 1"X 1" SQUARE PLUG
15	12470486	CAP, 3/4"X 1 1/2" RECTANGULAR PLUG
16	11013810	PENDANT, 6 FUNC, EASYCARE PLUS
17	39001169	KIT, PENDANT PATCH CORD, STAFF CONTROL (USED ONLY WITH PENDANT ITEM #16)


NOTE: ADD "AL" SUFFIX TO PART NUMBERS FOR SOFT TONE PAINTED PARTS.



MATTRESS SUPPORT
PLATFORM COMPONENTS
80" FRAME
MODELS: CARE 100

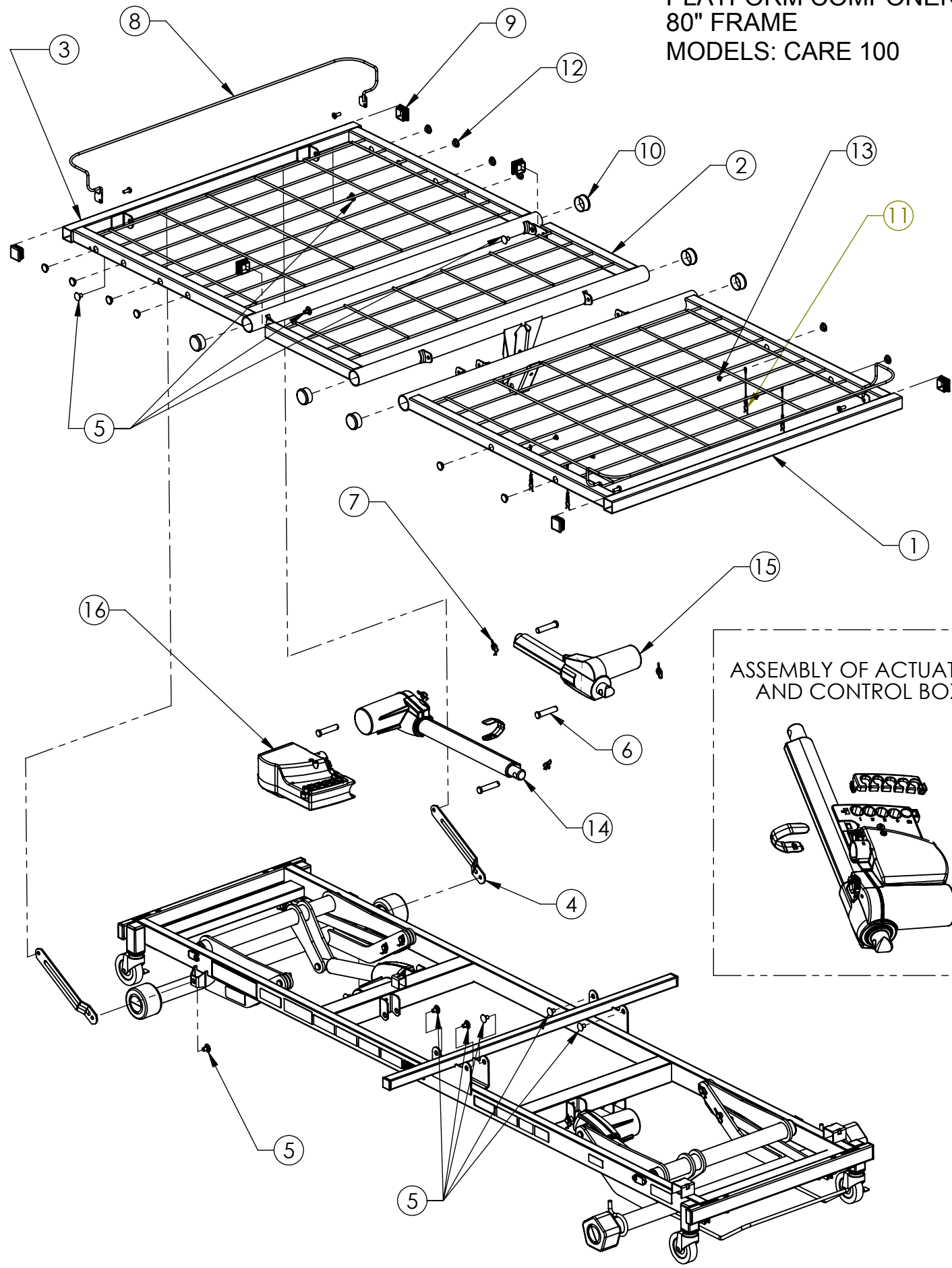

ASSEMBLY OF ACTUATOR
AND CONTROL BOX

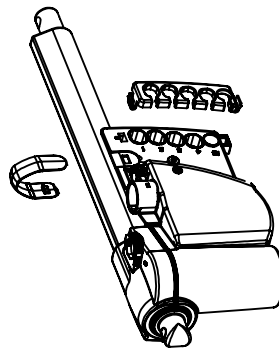


MATTRESS SUPPORT
PLATFORM COMPONENTS
80" FRAME
MODELS: CARE 100

ITEM NO	PART NUMBER	DESCRIPTION
1	24006385	WELDMENT, BACK SECTION, 80 INCH
2	24006387	WELDMENT, KNEE SECTION
3	24006386	WELDMENT, FOOT SECTION
4	11010381	LEVER, LEG
5	39000052	KIT, RIVET, REPLACEMENT
6	11012387	PIN, CLEVIS HEADED
7	12800032	RING, RUE-22
8	39001170	KIT, MATTRESS STOP
9	12470477	CAP, 1 1/4", SQUARE PLUG
10	12470474	CAP, 1 1/2", ROUND PLUG
11	24005271	CLIP ASSEMBLY
12	39001067	KIT, SIDE RAIL PLUGS, ECS
13	12551015	SCREW, PPHTF, 10-32 X 1/4"
14	39001090	KIT, ACTUATOR, BACK
15	39001020	KIT, ACTUATOR, KNEE
NS	11012397	MOTOR CABLE, KNEE ACTUATOR, BLUE O-RING
NS	11012469	MOTOR CABLE, HEAD ACTUATOR, YEL O-RING
16	39000852	KIT, CONTROL BOX

NOTE: ADD "AL" SUFFIX TO PART NUMBERS FOR SOFT TONE PAINTED PARTS.

ROLL IN THE LOW POSITION -
STANDARD CONFIGURATION
80" FRAME

MODELS: CARE 100
ROLL IN LOW

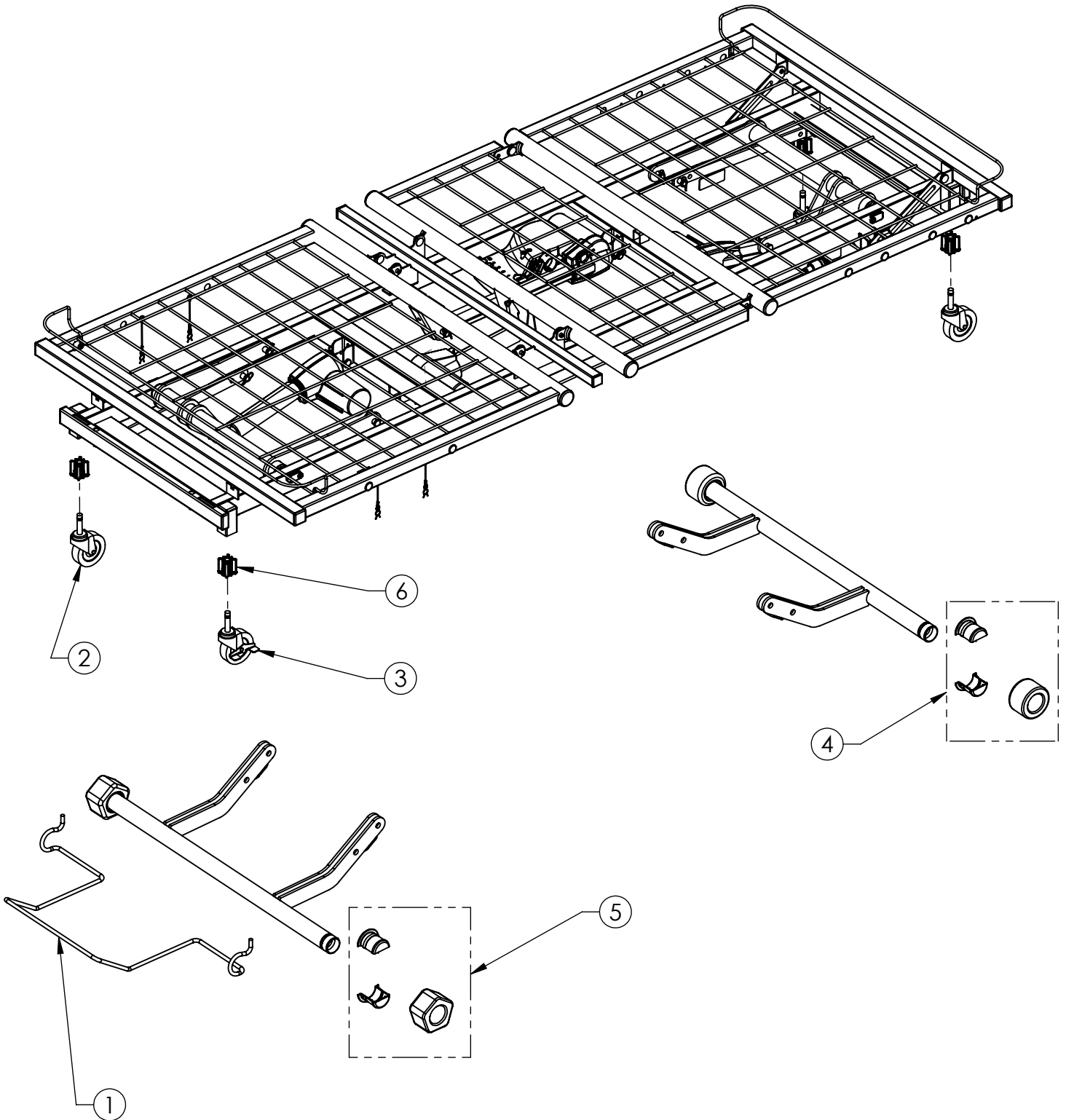


ROLL IN THE LOW POSITION -
STANDARD CONFIGURATION
80" FRAME
MODELS: CARE 100
ROLL IN LOW

ITEM NO	PART NUMBER	DESCRIPTION
1	11011867	BUMPER, WALL
2	10630301	CASTER, SWIVEL
3	10630305	CASTER, LOCKING
4	39000792	KIT, ROLLER AND HUB
5	39000793	KIT, PENTAGON AND HUB
6	11009561	CASTER SOCKET

NOTE: ADD "AL" SUFFIX TO PART NUMBERS FOR SOFT TONE PAINTED PARTS.



ROLL AT ANY HEIGHT -
OPTIONAL CONFIGURATION
80" FRAME
MODELS: CARE 100
ROLL AT ANY HEIGHT

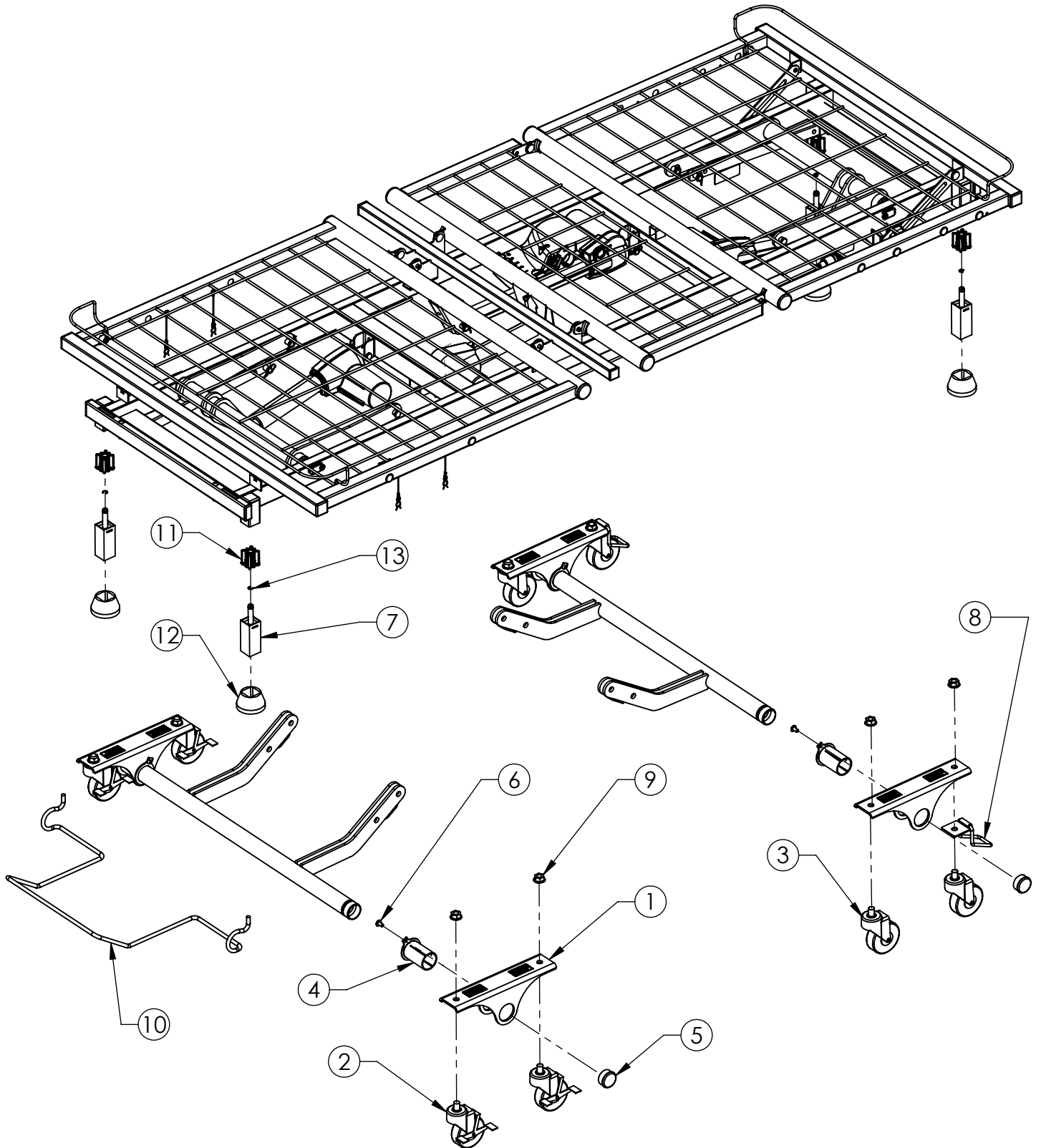


ROLL AT ANY HEIGHT -
 OPTIONAL CONFIGURATION
 80" FRAME
 MODELS: CARE 100
 ROLL AT ANY HEIGHT

ITEM NO	PART NUMBER	DESCRIPTION
1	11012492	BRACKET, CASTER
2	12630001	CASTER, LOCKING
3	12630000	CASTER, SWIVEL
4	11012241	BEARING, GRIPLOCK, ASSY., PIVOT
5	12470474	CAP, PLUG, 1 1/2 DIAMETER, BLACK
6	12552586	SCREW, 1/4 - 20 X 1/2, TX TRS HD
7	24005617	WELDMENT, STEM ASSM
8	31008901	WHEEL LOCK, STRAIGHT
9	12675024	NUT, 1/2-13, WHIZ LOCK, ZINC
10	11011867	BUMPER, WALL
11	11009561	CASTER SOCKET
12	11009576	FOOT PAD
13	11009562	RING GRIP


NOTE: ADD "AL" SUFFIX TO PART NUMBERS FOR SOFT TONE PAINTED PARTS.

STAFF CONTROL COMPONENTS
80" FRAME
MODELS: CARE 100

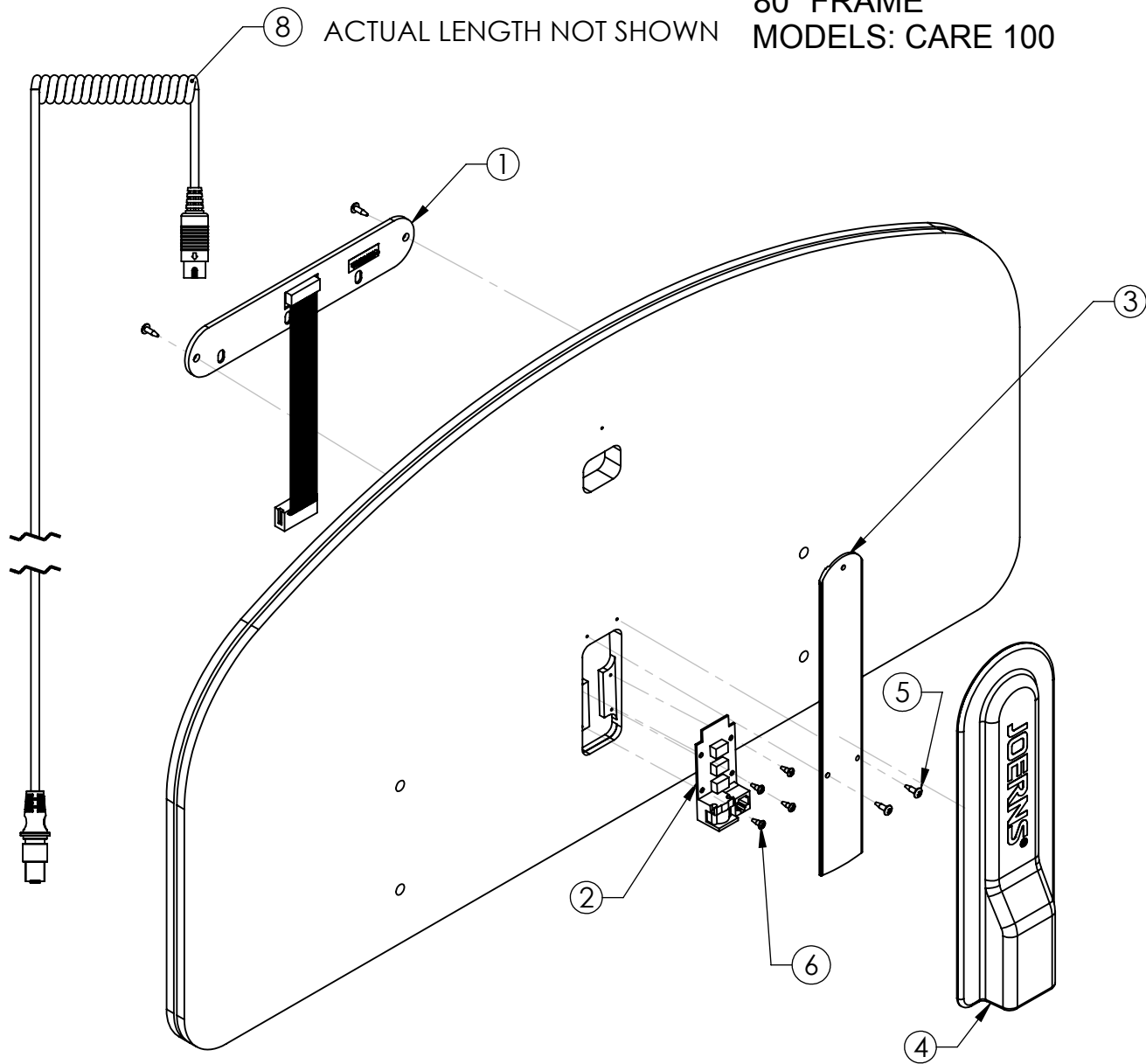

ITEM NO.	PART NUMBER	DESCRIPTION
1	11012068	STAFF CONTROLLER
2	11011775	PC BOARD, DC STAFF CONTROL
3	11012206	COVER, WIRE
4	11012145	COVER, PC BOARD
5	12570626	SCREW, #6 X 1/2 PTHSMS
6	12570618	SCREW, #6 X 3/8 PPHSMS
7	39000860	KIT, WIRE COVER (NOT SHOWN)
8	11013638	PATCH CABLE, STAFF CONTROL

PANEL COMPONENTS
80" FRAME
MODELS: CARE 100

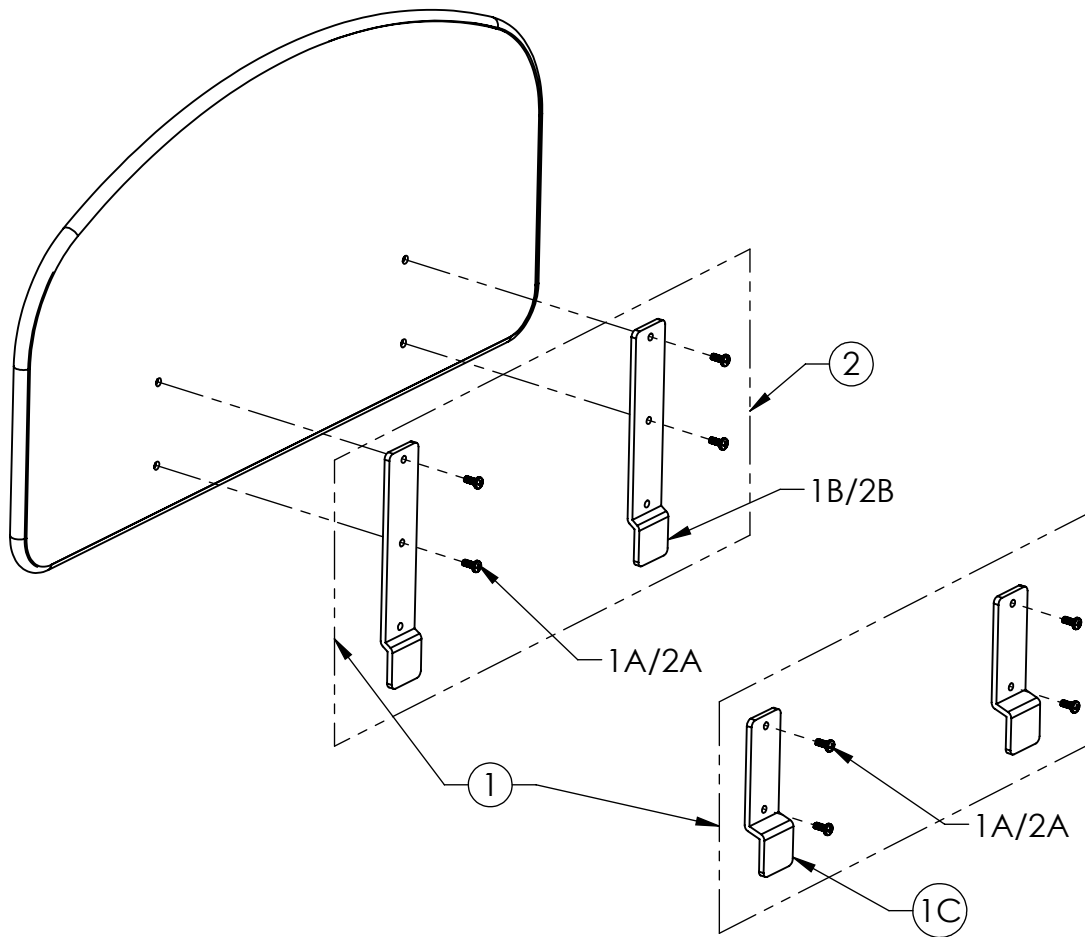

ITEM 1 35000713AL HARDWARE, HEAD AND FOOT PANELS		
PART	PART #	DESCRIPTION
1A	12542520	SCREW, 1/4-20 X 5/8
1B	11012848	BRACKET, HEAD PANEL
1C	11012724	BRACKET, FOOT PANEL
NS	6150316	INSTALLATION INSTRUCTIONS

ITEM 2 35000714AL HARDWARE, HEAD PANEL ONLY		
PART	PART #	DESCRIPTION
2A	12542520	SCREW, 1/4-20 X 5/8
2B	11012848	BRACKET, HEAD PANEL
NS	6150316	INSTALLATION INSTRUCTIONS

NOTE: ADD "AL" SUFFIX TO PART NUMBERS FOR SOFT TONE PAINTED PARTS

"NS"= NOT SHOWN



Post Acute, Acute, HomeCare
2430 Whitehall Park Dr. Ste 100
Charlotte, NC 28273
(P) 800.826.0270
(F) 800.457.8827

VA/Government
19748 Dearborn Street
Chatsworth, CA 91311
(P) 800.966.6662
(F) 800.232.9796

Canadian Office
1000 Clarke Rd. Ste 6
London, ON Canada N5V 3A9
(P) 866.546.1151
(F) 519.451.8662

**United Kingdom and
Other Countries**
+44 (0)844 811 1156
+44 (0)844 811 1157

Netherlands
+31 (0)30 6363700
+31 (0)30 6363799